

ePatch®

Optimizing your ECG solution with easy, continuous health monitoring

RITHM

- Continuous health monitoring – 24 hours a day, 5 days (120 hours)
- Manually mark events – Records symptomatic episode occurrence time
- Water-resistant IPX4 (splash proof) – Gently shower without removing the device
- Convenient for the patient to wear
- No lead wires – Lightweight, comfortable to wear and adaptable to a patient's lifestyle
- Easy to send
- Clinically validated – FDA Cleared and CE Marked



Step 1

Shave and clean the skin



Step 2

Insert sensor into patch



Step 3

Peel off back adhesive cover, attach patch to skin and peel off front paper cover

Technical Specifications

Placement:	Torso
Recording time:	Up to 120 hours
Sampling rate:	128/256/514 or 1024 Hz
Resolution:	12 or 16 bit
Storage medium:	2 GB internal storage
Weight:	16 g
Water resistance:	IPX4 (splash proof)
Real time clock:	Yes
Event marker function:	Yes, via double tab on sensor
Data acquisition:	1, 2, or 3 channels ECG, Event Marker